


(Drawing is not to scale)

Fabric Cuts

A+B (Width x Length):
7x8" for Large
7x7" for Medium
7x6" for Small
7x5" for child


C+D (Width x Length):
3x4" for all sizes

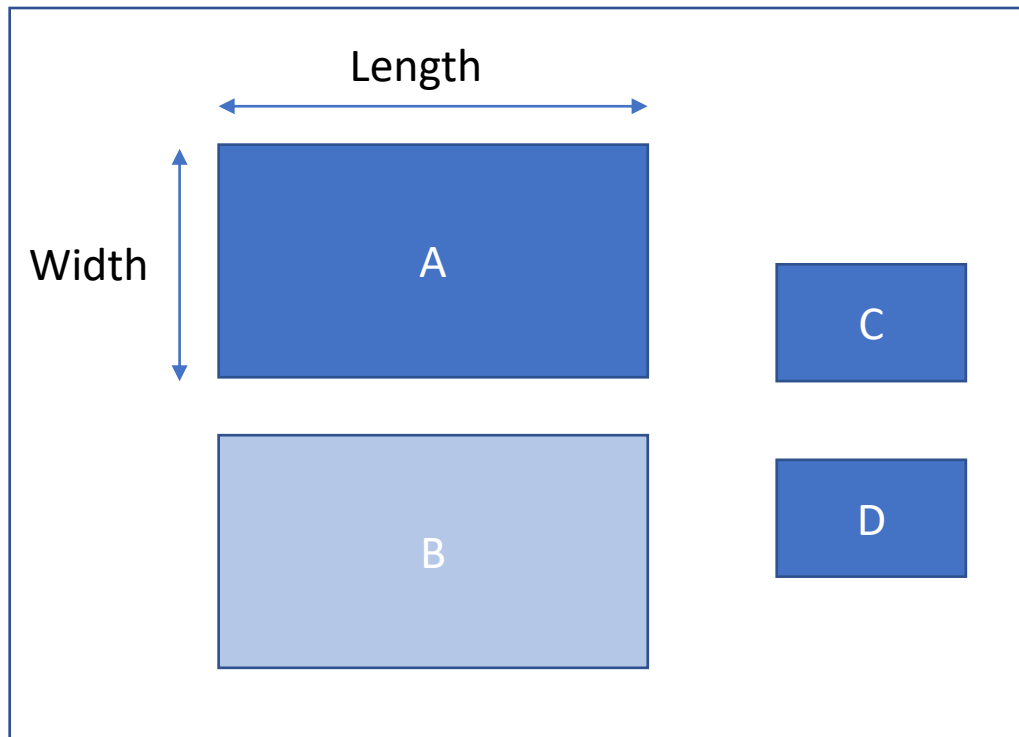

Finished Sizes


As folded (Width x Length):
3x9" for Large
3x8" for Medium
3x7" for Small
3x6" for child

As expanded (Width x Length):
6.5x9" for Large
6.5x8" for Medium
6.5x7" for Small
6.5x6" for child

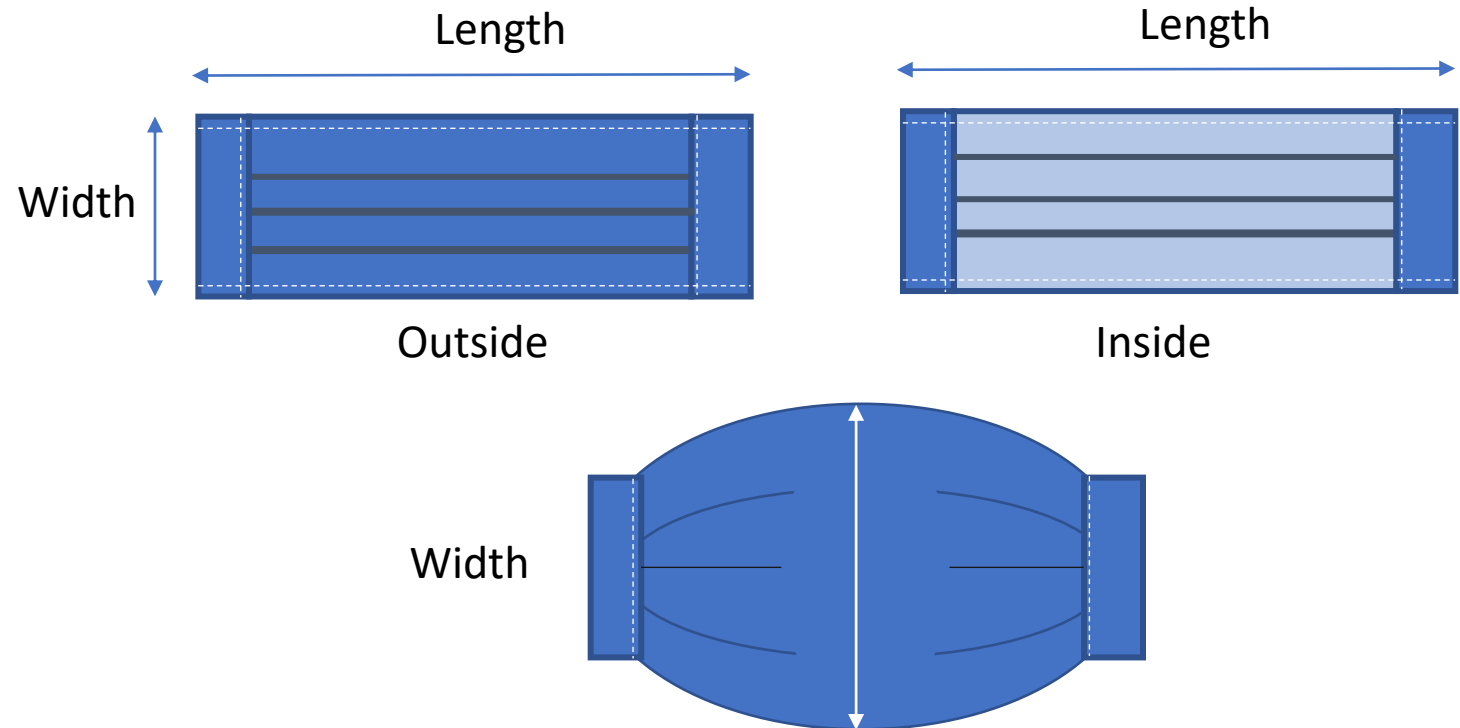


Step 1 – Put main pieces together


- Put the front sides of the fabrics face each other.
- Sew along the length side (not the 7" side) of the seams together at $\frac{1}{4}$ " from the edges.
(If you open it up, it form a tube).

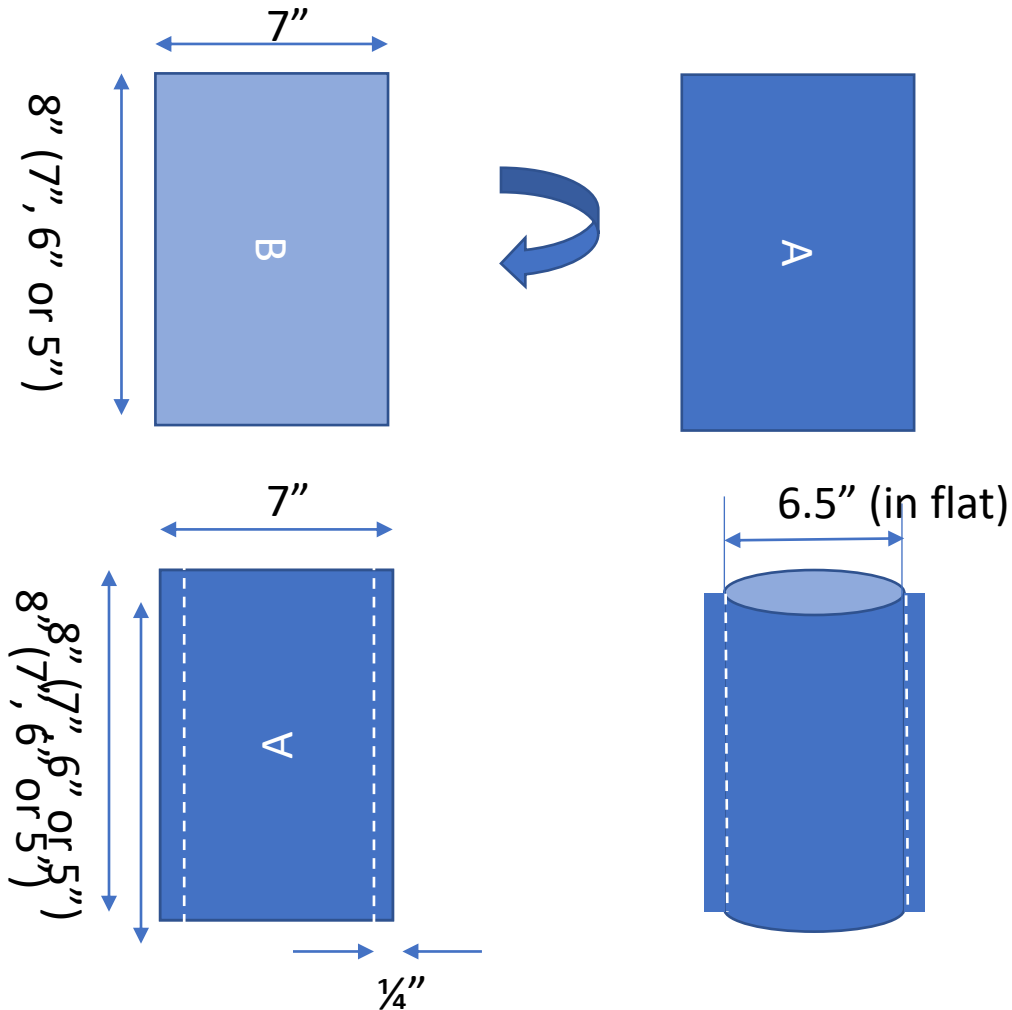

- Turn the tube inside out so the right side of the fabrics on the outside. Press (optional)
- Sew along the seam again at $\frac{1}{4}$ " away from the edges.

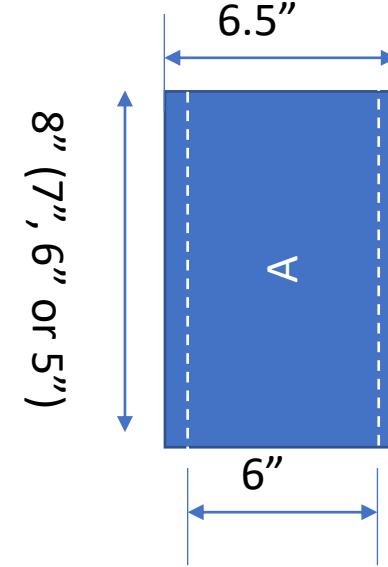


c. Make pleats

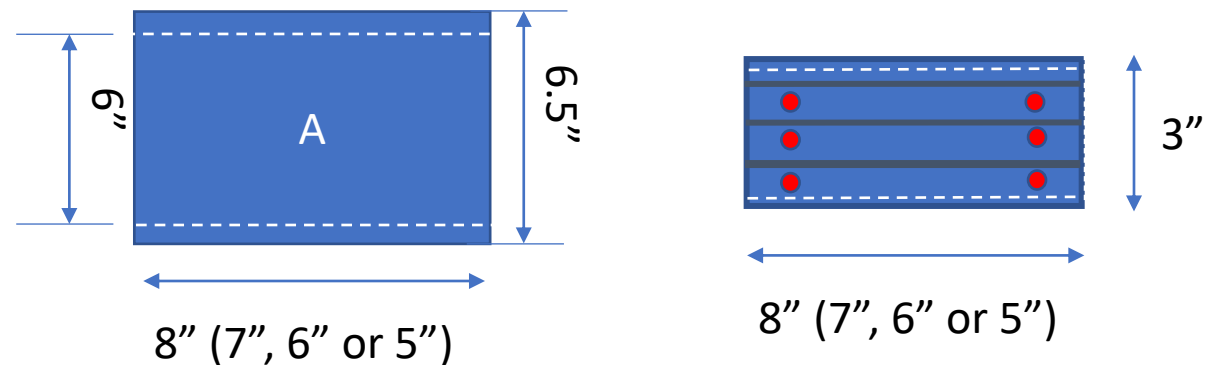

Step 2 – Make the bias for the edges


(both C and D are the same procedure. Assume back side this blue color fabric is yellow color)

- Fold $\frac{1}{2}$ " on each end of the 4" side.
- Sew $\frac{1}{4}$ " from the edges.
- Fold over the un-sewed sides $\frac{1}{2}$ " in and press.
- Fold it again in $\frac{1}{2}$ and press

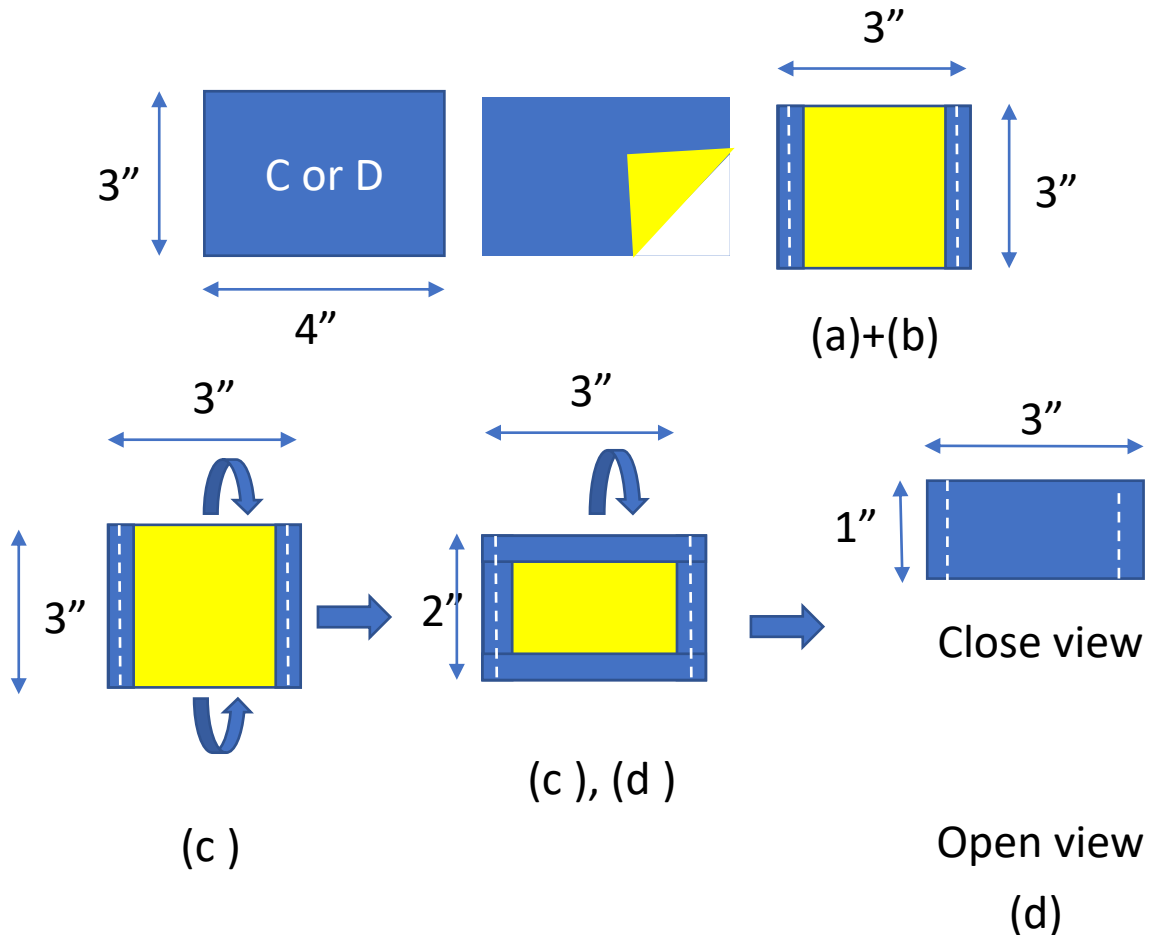

Step 3 – Assemble


- Put the edge bias made in Step 2 at the edge of pleated center piece from Step 1.
- The bias will cover $\frac{1}{2}$ " of the pleated edge.
- Sew $\frac{1}{4}$ " or less from the bias open edge.
- Do the other side

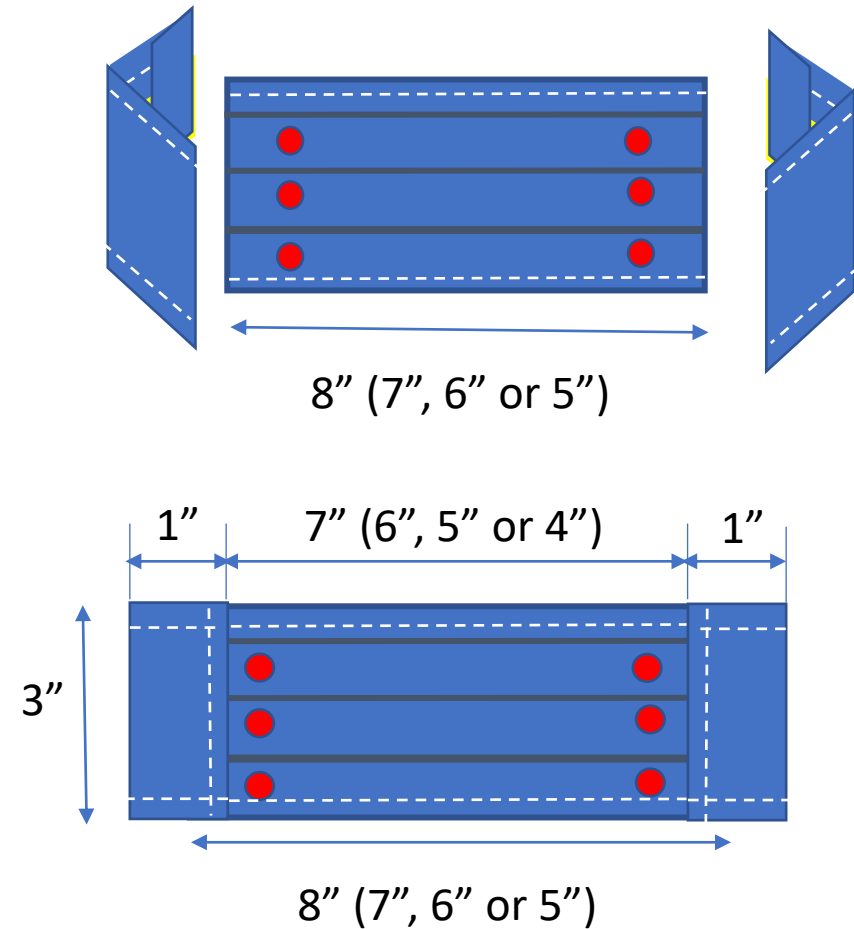

Step 4 – Add elastics or string

Ear loop type-

Elastic cut length: 10" for all sizes

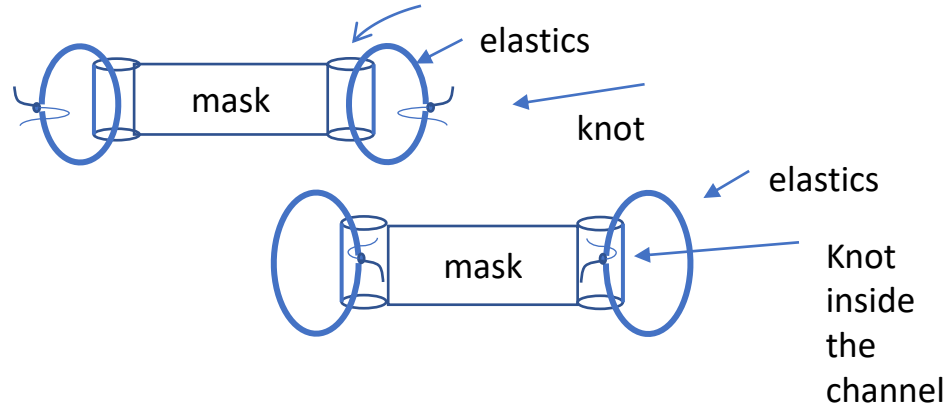

Over head type –

Elastic cut length:

29" for large

27" for med

25" for small

23" for child


Non-elastic string length:

40" for large

38" for med

36" for small

34" for child

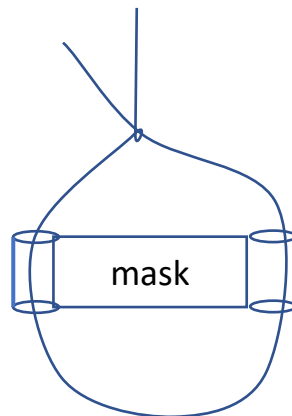

WASH THE MASK BEFORE USE!!

Wash as often as you can to avoid bacteria built up. They are not N95 masks. They are close substitute for surgical masks. You may wash with soap and water, then put in microwave (~2 min) to dry and disinfect if mask does not come with nose wire.

The mask needs to fit your face tightly. If the mask does not fit tightly on your face, please adjust the elastic or string length. For ear loop type of masks, you may use ear saver provided by 3DC19.

Donation is appreciated. We suggest \$3/mask or whatever you feel is right. You may leave cash or donate online at www.3DC19.com.